

# clinell<sup>®</sup> clean




**Clinell Clean Indicator Tape** is a convenient way to help implement effective hygiene procedures. Ideal for commodes, bed frames, drip stands and other equipment; the tape should be attached after disinfection to show that the object has been sanitised. It should be filled out to demonstrate the time, date and name of the member of staff who has disinfected the equipment.

## Product Highlights


- 1) Over £2.00 cheaper than other indicator tapes
- 2) 100m in length - 85 applications
- 3) Large 2cm adhesive strip - extra firm hold
- 4) Date, time & name boxes provided
- 5) Pen included with each roll




## Instructions for use

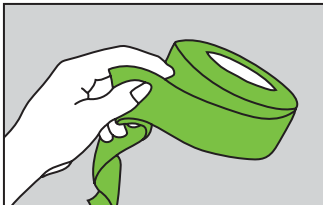


Select a suitable length of tape and tear through the roll so that you have a strip of indicators.

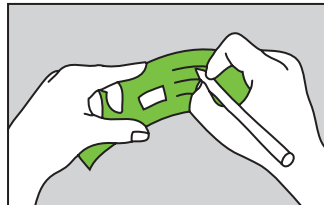

Clearly tick the 'I am clean' box and write your name, the time and the date in the area provided.

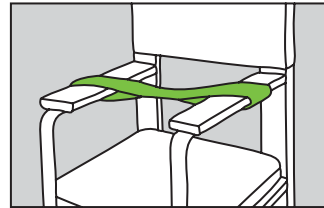

Loop the strip across the object, ensuring the coloured side is facing out and the dated strip is visible.

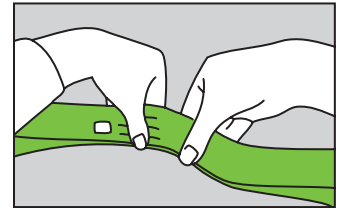

To join the ends of the tape, firmly press the two sides together and rub across, to bind the sticky underside.

## How to Order

Description	NHS Supply Chain	Bunzl	Price inc. VAT
Single Roll	FSE119	Z6CCIT100	£3.99
Case of 12	FSE119 x 12	Z6CCIT100 x 12	£3.94



GAMA Healthcare Ltd. Customer Services, 39 Hampstead House, 176 Finchley Road, London NW3 6BT, UK.  
Tel: +44 (0) 845 2011 644 [www.clinell.com](http://www.clinell.com) [info@gamahealthcare.com](mailto:info@gamahealthcare.com)

**SWITCH & SAVE  
OVER £2.00  
PER ROLL**